V9880

EN

The hot/cold pack provides convenient, reusable Cold of Hot treatment therapy. It will remain flexible even when cold, and can be applied to any parts of the body. When properly used, especially within the first 72 hours after injury, the pack helps reduce the severity of symptoms and the time required for healing.

To use cold

Store in freezer, remove and wrap in bag or cover. Apply to affected area. A minimum of 2 hours is required in the freezer to obtain the test results.

DO NOT USE DIRECTLY ON THE SKIN STRAIGHT FROM THE FREEZER.

Please make sure that the pack is placed in a cover.

To use hot

Hot water use:

1. Make sure to boil enough water to cover the pack completely
2. Remove the water from the heat
3. Place the pack in the water for about 7 minutes, making sure pack is completely covered by water
4. Leave it for about 10 minutes if pack is taken from freezer
5. Remove from the water and wrap in a towel or cover, apply to affected area
- 6.

Microwave use:

Be cautious as microwaves vary in power and you will need to find the correct amount of time to heat the pack.

1. Flatten the pack to ensure the gels is evenly distributed
 2. Place the pack in microwave oven and set timer for a max of 10 seconds and then leave it for about 30 seconds.
 3. If more heat is required, heat in 10 seconds increments with equal standing time until desired temperature is achieved
 4. Remove from microwave and wrap in a towel or cloth and apply to desired area
- NOTE: the medium power is recommended for the microwave over. Do not heat in microwave over 1400 WATT

Caution:

- ❖ If you suffer from neurovascular disease please consult your doctor before use
- ❖ When using for treatment of post- surgical trauma, follow instructions of your medial practitioner
- ❖ Please don't overheat the pack in the microwave beyond the recommended time. This may causes leakage of contents
- ❖ If contents are exposed to the skin or eyes, flush with water immediately. Gel is Non-Toxic
- ❖ If irritation or redness persists, consult a physician
- ❖ To be used under adult supervision
- ❖ The gel is inedible

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